

## PT/INR Dade® Innovin® Reagents

Reagent	Prep	Placement	On-Board Stability	Refrigerated Stability (2-8°C)
Dade® Innovin	Reconstitute with volume of de-ionized water specified on bottle, swirl to mix, allow to clear	Reagent holder position 1* (GW5 vial)	24 hours	10 days
Dade® Ci-Trol® 1	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	16 hours
Dade® Ci-Trol® 3	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	16 hours
CA Clean® I	Ready to Use	Rinse position 11 (PV-10 or SLD vial)	24 hours	1 month

\*Alternatively, any other cooled reagent position can be used.

## aPTT Dade® Actin® FSL Reagents

Reagent	Prep	Placement	On-Board Stability	Refrigerated Stability (2-8°C)
Dade® Actin® FSL	Mix by gentle inversion (5-8 times)	Reagent holder position 2* (GW5 vial)	48 hours	7 days
CaCl <sub>2</sub>	Ready to Use	Reagent holder position 7 <sup>a</sup> (GW5 vial)	48 hours	8 weeks
Dade® Ci-Trol® 1	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	16 hours
Dade® Ci-Trol® 3	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	16 hours
CA Clean® I	Ready to Use	Rinse position 11 (PV-10 or SLD Vial)	24 hours	1 month

\*Alternatively, any other cooled reagent position can be used.

<sup>a</sup>Alternatively, any other non-cooled reagent position can be used.

## Fibrinogen Dade® Thrombin Reagents

Reagent	Prep	Placement	On-Board Stability	Refrigerated Stability (2-8°C)
Dade® Thrombin	Reconstitute with volume of de-ionized water specified on bottle, swirl to mix, allow to clear	Reagent holder position 3* (GW5 vial)	24 hours	5 days
Dade® OVB	Ready to use at room temperature	Buffer Position 12 (GW 5, PV-10, SLD Vial)	8 hours	8 weeks
Dade® Ci-Trol® 1	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	16 hours
Data-Fi® Abnormal Fibrinogen Control	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	16 hours
Standard Human Plasma	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack (position 1)	n/a	n/a <sup>a</sup>
CA Clean® I	Ready to Use	Rinse position 11 (PV-10 or SLD Vial)	24 hours	1 month

\*Alternatively, any other cooled reagent position may be used.

<sup>a</sup>Standard Human Plasma is only used for calibration and should be kept at room temperature once reconstituted. It should be used within 4 hours.

## INNOVANCE® D-Dimer Reagents

Reagent	Prep	Placement	On-Board Stability	Refrigerated Stability (2-8°C)
INNOVANCE® D-Dimer Reagent	Reconstitute with 4mL de-ionized water, let stand for 15 mins, swirl to mix	Reagent holder position 6 <sup>a</sup>	16 hours	4 weeks
INNOVANCE® D-Dimer Buffer	Ready to Use	Reagent holder position 4 <sup>*</sup>	16 hours	4 weeks
INNOVANCE® D-Dimer Supplement	Ready to Use	Reagent holder position 8 <sup>a</sup>	16 hours	4 weeks
INNOVANCE® D-Dimer Diluent	Ready to Use	Reagent holder position 10	16 hours	4 weeks
INNOVANCE® D-Dimer Control 1	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	7 days
INNOVANCE® D-Dimer Control 2	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	n/a	7 days
INNOVANCE® D-Dimer Calibrator	Reconstitute with 1mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack (position 1)	n/a	n/a <sup>b</sup>
Dade® OVB	Ready to use at room temperature	Buffer position 12 (GW 5 or SLD Vial)	48 hours	8 weeks

<sup>\*</sup>Alternatively, any other cooled position can be used.

<sup>a</sup>Alternatively, any other non-cooled position can be used.

<sup>b</sup>INNOVANCE D-Dimer Calibrator is only used for calibration and should be kept at room temperature once reconstituted. It should be used within 4 hours.

In original vials, the reagents may be left on board the instrument continuously for 16 hours or stored on and off the instrument for intervals of 7 x 1 hour over a maximum period of 14 days.

## INNOVANCE® Heparin Reagents

Reagent	Prep	Placement	On-Board Stability	Refrigerated Stability (2-8°C)
INNOVANCE® Heparin Reagent	Ready to Use	Reagent holder position 1*	16 hours	8 weeks
INNOVANCE® Heparin Substrate	Ready to Use	Reagent holder position 2*	16 hours	8 weeks
Dade® OVB	Ready to use at room temperature	Buffer Position 12 (PV-10)	16 hours	8 weeks
INNOVANCE® Heparin LMW Control 1	Reconstitute with 1 mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	8 hours	48 hours
INNOVANCE® Heparin LMW Control 2	Reconstitute with 1 mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	8 hours	48 hours
INNOVANCE® Heparin UF Control 1	Reconstitute with 1 mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	8 hours	48 hours
INNOVANCE® Heparin UF Control 2	Reconstitute with 1 mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack	8 hours	48 hours
INNOVANCE® Heparin Calibrators 1-5	Reconstitute with 1 mL de-ionized water, let stand for 15 mins, swirl to mix	Sample cup placed in a Sample Rack (positions 1-5)	4 hours	48 hours
CA Clean® I	Ready to Use	Rinse position 11 (PV-10)	24 hours	1 month

\*Alternatively, any other cooled reagent position can be used.

Dade, Innovin, Actin, INNOVANCE, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc. or its affiliates. All other trademarks and brands are the property of their respective owners. Sysmex is a registered trademark of Sysmex Corporation of America. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

**Note: This document is for supplemental use only, and not meant to be used in place of primary technical materials.**

**This document is copyrighted. No part of this may be copied, reproduced, translated, or reduced to any electronic medium or machine-readable form without the prior written consent of Siemens Healthcare Diagnostics, Inc.**

---

**Siemens Healthineers Headquarters**

Siemens Healthcare GmbH  
Henkestr. 127  
91052 Erlangen  
Germany  
Phone: +49 9131 84 0  
[siemens.com/healthineers](https://www.siemens.com/healthineers)

**Local Contact Information**

Siemens Healthcare Diagnostics Products GmbH  
Laboratory Diagnostics  
Emil-von-Behring-Str. 76  
35041 Marburg  
Germany  
Phone: +49 6421-39-3061  
[siemens.com/healthineers](https://www.siemens.com/healthineers)